transcom

TRANSCENDENCIAS COMERCIALES S.L.

Parque Empresarial Zuatzu C/Zubiberri nº31
Edif. Urumea Finca 1 - Planta baja - Local 1
20018 San Sebastián (Guipúzcoa) ESPAÑA
Tel.: +34 943 224 360 Fax: +34 943 224 275
www.transcomsl.com | e-mail: info@transcomsl.com

B482Z090

111787641737

HYDROKIT WITH LIMITER

Z-17

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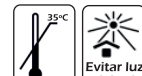
ES

TRADE NAME

HYDROKIT WITH LIMITER | MADE IN SPAIN

CONTENTS

Cannula with limiter 6 + Obturator + Water line (1500 mm / 59") + Waste Hose (1300mm / 51")

All components are free of phthalates

SINGLE USE ONLY

Under no circumstances should any item of this kit be reused, as this could lead to blood-borne and venereal diseases, or via contact with faeces.

CAUTION

Risk of perforation if instructions are not followed. Not to be used on children or pregnant women. Only for use by Healthcare Professionals. This device is not intended for lay users. This kit is to be used only with TRANSCOM Colon Cleansing devices (Z-01 HC-1 CLASSIC, Z-02 HC-2000 CLASSIC, Z-04 HC-1, Z-05 HC-2000). Before using the Hydrokit, please also refer to the Contraindications and Side Effects of the active devices. The device would be used in a professional office (medical office type equipment) environment. Caution: Federal law restricts this device to sale by or on the order of a physician.

INSTRUCTIONS

Connect water line to cannula, and insert obturator in the cannula. Lubricate the cannula and insert it gently into the rectum. Then remove the obturator and attach the waste hose to the cannula. After each treatment, dispose of the entire kit in a secure way to avoid contamination.

RESPONSIBILITY

The customer will be entirely responsible if these instructions and indications for use supplied by TRANSCOM SL are not followed.

INTENDED PURPOSE

Through its components and by means of the connection between the active device and the patient, the Hydrokit is meant to support the intended purpose of the active devices, the cleaning action is carried out solely through the latter.

TARGET PATIENT GROUPS

Patients who require radiological or endoscopic examination.

INTENDED USERS

Healthcare Professionals.

EXPECTED CLINICAL BENEFITS

By means of a colonic irrigation, it lets a cleansing of the colon through the connection between the active device and the patient. Biocompatible speculum material, does not cause allergies or irritation. Consistent quality throughout its shelf life, drip-free connections, airtight bag seal. Microbial load on the surface of the speculums controlled to avoid potential infections. Hygienic cleaning procedures, single use kits. Ergonomic shape and adaptable characteristics to population's anatomy

PERFORMANCE CHARACTERISTICS

- Easy to handle, the speculum has a flap and the obturator has a ring-shaped design.
Speculum
- Convex-Curve shape at the tip for easy insertion.
- Ergonomically designed to stay in place during treatment.
- Ring design for a secure fit on Water line and Waste hose.
Obturador
- Round tip obturator for comfortable insertion.

SAFE DISPOSAL OF THE PRODUCT

After use, the components must be handled according to universal standard precautions, as they will be contaminated. Specific bags sufficiently resistant for the disposal of these wastes must be available, which must be in puncture and breakage-proof containers and shall be closed and managed as sanitary waste assimilable to urban waste.

CONTRAINDICATIONS:

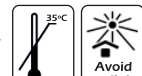
Wounded, breached or damaged skin in contact with the device.

SIDE-EFFECTS

No side-effects have been observed in the use of the device.

Any serious incident related to the product must be reported to the manufacturer and the health authority.

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